

## Conception Kit™ INSTRUCTION MANUAL

Conception is not an exact science. From the sperm to the egg and everything in between, you may have a number of reasons that can cause conception not to occur. The Conception Kit™ was designed to address some of the problems that might be keeping a couple from conceiving. If you have any questions regarding conception, please consult your health care professional.

### WARNING



**Read and follow instructions in Conception Kit™ before use.**

### WARNING

**Use of Conception Kit™ is intended to increase the likelihood of becoming pregnant through assisted reproductive technology. Pregnancy may involve serious health risks for both you and your unborn child. Consult your health care professional for possible health risks before use of the Conception Kit™.**

If you do not understand these instructions or if you have any questions, log onto [www.conceptionkit.com](http://www.conceptionkit.com) for more information or to email Conceivex Inc. You can write to Conceivex Inc. @ Conceivex Inc., P.O. Box 31, Saranac Michigan 48881 USA.

Distributed by:

*Conceivex, Inc.*

Saranac, Michigan 48881 USA



## TABLE OF CONTENTS

	Page
1. What Is the Conception Kit™?	4
2. What Is Included in the Conception Kit™?	5
3. Safety Information - General Warnings and Cautions	6
4. Conception Kit™ Components	7
5. What You Should Know Before Using the Conception Kit™	9
6. How to Use the Conception Kit™ - 16 Steps	9
7. Care for the Conception Kit™	24

## WHAT IS THE Conception Kit™?


The Conception Kit™ is intended to increase the likelihood of becoming pregnant through assisted reproductive technology (ART). The Conception Kit™ is an innovative over-the-counter product that moves the sperm closer to the egg by placing the semen/sperm at the opening of the cervical os (the passage between the vaginal cavity and the uterus). The Conception Kit™ is a complete method designed to go beyond timing your ovulation, enhancing a couple's opportunity to conceive. The most important component of the Conception Kit™ is the Conception Cap. When the Conception Cap is used in conjunction with the other components in the Conception Kit™, the Conception Cap can address many of the problems that keep couples from conceiving. The Conception Cap is also referred to as a cervical cap with the intended purpose of conception.

**On the male side...**the Conception Kit™ may help address common difficulties such as:

- Motility issues with sperm
- Diminished sperm count
- Position of the penis opening (hypospadias and epispadias)

**On the female side...**the Conception Kit™ may help address common difficulties such as:

- Timing of ovulation
- Hostile vaginal environment
- Position during intercourse
- Tilted cervix

 For a more detailed description of these conditions refer to the Conception Kit™ Journal Section, "[How the Conception Kit™ Was Designed to Help](#)".

## WHAT IS INCLUDED IN THE Conception Kit™?

In the Conception Kit™, you will find:

- **3 Conception Caps**
- **24 Ovulation Predictors**
- **3 Pregnancy Test Kits**
- **3 Non-latex Semen Collectors**
- **3 Sperm-Friendly Intimate Moisturizer Samples**
- **1 Conception Journal**
- **1 Instruction Manual**
- **2 Conception Wheels (1 English and 1 French/Spanish)**
- **1 Medical Provider Note/Envelope**

*These components are designed for use in the privacy of your own home.*

## SAFETY INFORMATION



**There are important warnings and precautions in this manual.**



This is the general warning sign. It is used to alert you to potential hazards. All safety messages that follow this sign must be obeyed to avoid potential harm.

### **WARNING**

: indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

### **CAUTION**

: indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.

## GENERAL WARNINGS AND CAUTIONS

### WARNING



Read and follow instructions in Conception Kit™ before use.

### WARNING

Use of Conception Kit™ is intended to increase the likelihood of becoming pregnant. Pregnancy may involve serious health risks for both you and your unborn child. Consult your health care professional for possible health risks before use of the Conception Kit™.

### WARNING

Conception Kit™ components will not protect against HIV or any other sexually transmitted diseases (STD's).

### WARNING

To avoid choking hazard, keep all Conception Kit™ components out of reach of children.



### WARNING

Conception Kit™ components are not intended for contraceptive use. Safety and effectiveness of the Conception Kit's® Cervical Cap for contraception have not been established.

### CAUTION

② Do not reuse any Conception Kit™ component.

**NOTE:**  Do not use any component of the Conception Kit™ after the expiration date.

**NOTE:** 2C  30 C (35 F  86 F) The Conception Kit™ can be stored at temperatures of between 2 and 30 degrees Celsius or 35 to 86 Fahrenheit . The Conception Kit™ should be kept away from direct sunlight, moisture and heat.

## Conception Kit™ COMPONENTS

The Conception Kit™ is intended to enhance your opportunity to conceive. In the Conception Kit™, you will find:

- **3 Conception Caps**

The Conception Cap is a flexible cuplike receptacle that fits over the cervix to aid in conception.

- **24 Ovulation Predictors**

The ovulation predictor is a device used to predict when the woman's egg will be released.

- **3 Pregnancy Test Kits**

The pregnancy test kit is a device used to indicate pregnancy.

- **3 Non-latex Semen Collectors**

The semen collector is a condom-like sheath that completely covers the penis with a closely fitting membrane. The collector is used to collect semen and is not a contraceptive.

- **3 Sperm-Friendly Intimate Moisturizer Samples**

If desired, the sperm-friendly intimate moisturizer sample may be used to help lubricate the vagina/vaginal passage.

- **1 Conception Journal**

The Conception Journal contains helpful information and two very important charts. The first chart is a diary/chart that should be completed daily while using the Conception Kit™. Charting will give you insight as to how your fertility cycle works. For example charting helps you identify: when the cycle starts, when ovulation is occurring, when the Conception Cap is to be used and when the pregnancy test was taken. The second chart inside the Conception Journal is the Medical Appointment page. On this page you will be able to record all medical appointments, the purpose of the appointment and the medication that was

prescribed. Charting will also provide accurate information for your health care professional if you need to go further in your quest to conceive.

- **2 Conception Wheels (1 English, 1 French/Spanish)**

The Conception Wheel helps you time a pregnancy by determining: (1) when to stop taking birth control pills, (2) when to stop using a barrier contraceptive method, (3) when to start taking vitamins, and (4) when to start using the Conception Kit™.

- **1 Medical Provider Note**

The Medical Provider Note is a pre-printed note to be sent to your health care professional prior to using the Conception Kit™, informing him or her that you are trying to become pregnant.

*These components are designed for use in the privacy of your own home.*

## WHAT YOU SHOULD KNOW BEFORE USING THE Conception Kit™

The Conception Kit™ cannot overcome some conditions which may prevent pregnancy, including, but not limited to:

- Endometriosis
- No sperm in semen (Azospermia)
- Scarred Uterus
- Blocked Fallopian Tube
- Luteal Phase Defect
- Polycystic Ovary Syndrome
- Low amounts of Follicle Stimulating Hormone
- Cervical Stenosis
- Genital Warts
- Age
- Polyps and Fibroids
- Other serious conditions

⚠ If you have any questions regarding these or any other fertility concerns, consult your health care professional.

## HOW TO USE THE Conception Kit™ - 16 STEPS

### STEP 1:

**CONSULT YOUR HEALTH CARE PROFESSIONAL BEFORE USING THE Conception Kit™.**

### **⚠ WARNING**

**Use of Conception Kit™ is intended to increase the likelihood of becoming pregnant. Pregnancy may involve serious health risks for both you and your unborn child. Consult your health care professional for possible health risks before use of the Conception Kit™.**



**NOTE:** On the Conception Wheel you will see numbers 1, 2, 3 in the "Birth Window" section of the wheel. You will also see 1, 2, and 3 in the "Use Kit" section of the wheel. These numbers correlate. For example, if you use the kit and become pregnant the first month you use it, then you will likely deliver on the first month of the birth window.

**Once you've found your birth window, follow the time circle (which lists the months of the year) to see when to:**

- Stop using the birth control pill (if currently used)
- Start using a barrier contraception method if you do not want to get pregnant sooner than your desired three month birth window.
- Start taking prenatal vitamins (consult with your health care professional to obtain a list of recommended prenatal vitamins and supplements).
- Stop barrier contraception.
- Start charting your menstrual cycle.
- Begin to use the Conception Kit™.

#### **STEP 4:**

**STOP TAKING BIRTH CONTROL PILLS DURING THE MONTH INDICATED ON YOUR CONCEPTION WHEEL (IF BIRTH CONTROL PILLS ARE BEING USED).**

On average it can take at least six months for all birth control chemicals to leave a woman's body and for the fertility cycle to return to normal.

#### **STEP 5:**

**START USING A BARRIER CONTRACEPTION METHOD DURING THE MONTH YOU STOP YOUR BIRTH CONTROL PILL.**

Conceivex suggests using a barrier contraceptive method prior to using the Conception Kit™ while your body and hormones return to normal after stopping the birth control pill. This is a suggestion and is not mandatory.

**STEP 6:  
START TAKING PRENATAL VITAMINS AND  
SUPPLEMENTS DURING THE MONTH INDICATED ON  
YOUR CONCEPTION WHEEL.**

Conceivex recommends that one month prior to using the Conception Kit™, you start taking a multi-vitamin. Look for vitamins that contain B6, folic acid and iron. Over-the-counter prenatal vitamins are available.

**NOTE:** Consult with your health care professional to obtain a list of recommended prenatal vitamins and supplements.

**STEP 7:  
STOP USING BARRIER CONTRACEPTION DURING THE  
MONTH INDICATED ON YOUR CONCEPTION WHEEL.**

One month prior to using the Conception Kit™, stop using your barrier method of contraception.

**STEP 8:  
CHART YOUR CYCLE ON A DAILY BASIS.**

Filling out your chart on a daily basis is very important. Charting will give you insight as to when ovulation is occurring. Charting will also provide accurate information for your doctor or health care professional if you need to go further in your quest to conceive. It is important to keep track of the following:

- When your cycle started, the first day of your period
- Consistency/look/feel of your cervical mucus
- The days you had sex during the month
- What days you used the ovulation predictors
- The day your luteinizing hormone is released, which is commonly called the LH surge. (For more information regarding the LH surge please consult the Conception Kit™ Journal and the Conception Kit™ web site: [www.conceptionkit.com](http://www.conceptionkit.com).)
- When the Conception Cap was used

- When the Conception Cap was removed
- What day you used the pregnancy test

By charting all these factors over a three month time period, your doctor or health care professional and you will have a better understanding of your body and your fertility cycle.

### **STEP 9: CHART YOUR MENSTRUAL CYCLE.**

After you stop using the barrier method, begin charting your menstrual cycle in the Conception Journal on the first day of your next period. Recording your menstrual cycle will help you and your health care professional better understand your reproductive cycle. Record your menstrual cycle over the three month period that you use the Conception Kit™.

### **STEP 10: FILL OUT YOUR MEDICAL APPOINTMENT PAGE AFTER EVERY MEDICAL APPOINTMENT.**

The Conception Journal's medical appointment page is located at the end of the Conception Journal. On this page you will be able to record all medical appointments, the purpose of the appointment and the medication that was prescribed. It is important to keep a record of this information while you are trying to conceive.

### **STEP 11: FIND YOUR CERVIX.**

To use the Conception Kit™ correctly, you will need to be familiar with your cervix. Knowing where your cervix is located is extremely important for successful placement of the Conception Cap. Prior to the LH surge, when you will use the Conception Cap, it is time to take a self-guided exploration of your female anatomy.

 CAUTION

**To avoid infection, wash your hands before finding your cervix. Rinse your hands thoroughly of all soap. Soap kills sperm.**

You do not want to introduce germs into the vaginal cavity. That is why washing your hands prior to exploration is a good idea. Rinse your hands thoroughly; you also do not want to introduce traces of soap into your vagina that could be left on your hands, since soap is spermicidal which kills sperm!

**NOTE:** It may be helpful to use a mirror when you begin looking for your cervix.

### **How to find your cervix (STEP 11):**

1. Separate the lips of your vagina and insert two fingers. Your vaginal wall will surround your fingers. The vagina is on an upward angle so, as you reach up inside your vagina, you will find that your fingers are at an upward angle. If you are having trouble reaching all the way to the back of the vagina where the cervix is, try bringing your knees to your chest.
2. As your fingers reach the back of your vagina, you will notice a dip on the vagina's back wall – the wall closest to the buttocks. What you are feeling is the area where the semen normally pools after intercourse. Just before the end of the vagina are the cervix and the cervical os. The cervix will feel like a closed tulip or the tip of your nose. If you have given birth, your cervix may feel like a smile.
3. Now that you have found your cervix, it is important to know that the cervix changes position at different times of your menstrual cycle or with sexual excitement. Once you know what your cervix feels like, you will be able to locate it when you use the Conception Cap. (See STEP 15 to know how to use the Conception Cap.)
4. When you locate your cervix, be sure to inspect your cervical mucus (see STEP 12 to know what to look for in cervical mucus).

## **STEP 12: INSPECT AND CHART CHANGES IN YOUR CERVICAL MUCUS.**



**To avoid infection, wash your hands before finding your cervical mucus. Rinse your hands thoroughly of all soap. Soap kills sperm.**

After you stop using the barrier method, inspect your cervical mucus on a daily basis. The different textures and colors of the cervical mucus will help you understand where you are in your menstrual cycle and how close you are to ovulation. You will find the cervical mucus on the inside walls of the vaginal cavity and on/around the cervix. The following is a list of textures and colors of mucus to inspect and chart in your Conception Journal:

- Dry or no cervical fluid in the vagina: this usually occurs after a menstrual cycle has ended.
- Sticky cervical mucus: this mucus will feel like rubber cement and be sticky.
- Creamy cervical mucus: this mucus signals that your hormones are starting to kick in. This is the time to look for the next stage in mucus, which is clear cervical mucus.
- Clear cervical mucus: feels and looks like egg whites. Egg-white cervical mucus is the term used to describe the cervical mucus just before ovulation. During this stage of the cycle the mucus becomes very thin and the sperm is able to penetrate it easily.

Although inspecting and charting your cervical mucus is very helpful in determining proper timing, it is not mandatory.

### STEP 13: USE THE OVULATION PREDICTORS (OPK) TO PINPOINT WHEN TO CONCEIVE.


#### WARNING

To avoid choking hazard, keep all Conception Kit™ components out of reach of children.

#### WARNING

Conception Kit™ components are not intended for contraceptive use.

#### CAUTION

 Ovulation predictor is for in-vitro diagnostic use only. Not for internal use.

#### CAUTION


Certain medications or medical conditions including but not limited to Polycystic Ovarian Syndrome (PCOS), may affect the reliability of this test for predicting ovulation.

**LIMITATIONS OF ANY OVULATION PREDICTOR:** Certain medical conditions and medications may affect the accuracy of this test for predicting ovulation. These can include pregnancy, post-partum, menopausal symptoms, breast feeding, birth control pills, some fertility medications and polycystic ovarian syndrome (PCOS). Women with medically diagnosed fertility problems should ask their health care professionals if the product is suitable for them.



**NOTE:**  Do not use ovulation predictors after expiration date.

**NOTE:**  Do not reuse an ovulation predictors.

**NOTE:** Do not stream (urinate) past the arrow on the OPK.

**NOTE:**  The following is ONLY a summary of how to use the ovulation predictors. For more information, read and follow all instructions on the manufacturer ovulation predictor insert found in the box.

## Summary of how to use the ovulation predictors (OPK):

1. Start using the ovulation predictor on the day your cervical mucus is creamy – this will usually be around day 8 – 12 of your menstrual cycle. Please refer to the OPK chart in the OPK insert to determine when you should start testing.
2. Do not use first morning urine samples. The best time to test is between 10am and 8pm. You should test around the same time each day. It is important that you reduce your liquid intake about 2 hours before collecting the urine.
3.  Each test stick can only be used once.
4. Remove the Conceivex ovulation test device from the sealed pouch and use it as soon as possible.
5. Hold the handle of the OPK in one hand. Use the other hand to remove the cap. You will want to save the cap for the next step, just put it aside for now.
6. Point the absorbent tip of the OPK downward, place in urine stream for at least 10 seconds to thoroughly wet.
7. After 5 seconds re-cap the device and wait for color bands to appear. Positive results may be observed in as little as 40 seconds. However, to confirm negative results, the complete reaction time of 5 minutes is required. Do not read results after more than 30 minutes. If no results are present repeat the process the next day.
8.  Reading the predictor: refer to the manufacturer ovulation predictor insert on how to read the color bands to determine if you are ovulating.
9. Once the predictor strips indicate that your LH surge has occurred, you should ovulate within 24-48 hours. After you have detected your LH surge you should have sex after 24 hours but before 48 hours using the semen collector, sperm-friendly intimate moisturizer sample and the Conception Cap. Your two most fertile days begin once you detect your LH surge. Once you have detected your surge there is no need to continue testing with the ovulation predictors.

## STEP 14: USE THE SPERM-FRIENDLY INTIMATE MOISTURIZER SAMPLE AND SEMEN COLLECTOR

### WARNING

Do not use semen from a partner who has HIV or other sexually transmitted disease (STD).

### WARNING

Non-latex semen collector will not protect against HIV or any other sexually transmitted diseases (STD's).

### WARNING

Sperm-friendly intimate moisturizer sample will not protect against HIV or any other sexually transmitted diseases (STD's).

### WARNING

If lesions, open sores, viral warts or other abnormalities are noticeable on either party, avoid sexual intercourse and consult your health care professional.


### WARNING

To avoid choking hazard, keep all Conception Kit™ components out of reach of children.

### CAUTION

Only use non-latex semen collectors provided in the Conception Kit™. Do not use condoms. Condoms may contain latex or spermicide, which could kill sperm and impede your opportunity to conceive.

**NOTE:** To avoid damage to furniture, do not place semen collector, open foil package or intimate moisturizer sample bulb on surface where lubricant can leak out.

**NOTE:**  The following is ONLY a summary of how to use sperm-friendly intimate moisturizer sample. For more information, read and follow all instructions in the manufacturer sperm-friendly intimate moisturizer matchbox sample found in the semen collector box.

**Summary of how to use the semen collector and if desired, sperm-friendly intimate moisturizer sample:**

1. You should have sex no earlier than 24 hours but no later than 48 hours using the semen collector, sperm friendly moisturizer sample and the Conception Cap after you have detected your LH surge. Your two most fertile days begin once you detect your LH surge with the ovulation predictors.
2. If desired, apply the sperm-friendly intimate moisturizer sample approximately 15 minutes prior to sex. To apply, grip the applicator by the thick end (bulb) and shake the product down to the narrow tip. Twist off the square tab on the applicator. Insert the narrow end of the applicator deep into the vagina to coat the interior of the vagina and cervix. This can be done while standing, lying down or sitting (as if on a toilet). Press down on the thick end of the applicator several times to deposit the moisturizer. Remove and dispose of the applicator.
3. Have sex using the semen collector and if desired, the sperm-friendly intimate moisturizer sample.
4. Place the semen collector against the top of the erect penis. To prevent trapping air, hold the reservoir tip while unrolling it onto the penis.
5. After ejaculation, remove the semen collector, being certain not to lose any of the semen. The semen will be thicker and easier to work with when transferring it to the Conception Cap if this is done immediately.

The intimate moisturizer included in the kit is a sample for the convenience of the woman and couple

## STEP 15: USE THE Conception Cap.

### WARNING

Conception Cap will not protect against HIV, or other sexually transmitted diseases (STD's).

### WARNING

To avoid HIV or other sexually transmitted diseases (STD's). Do not share the Conception Kit™.

### WARNING

Remove the Conception Cap after 6 to 8 hours or after use overnight. Failure to remove the Conception Cap could result in blocked menstrual flow or infection.


### WARNING

To avoid choking hazard, keep all Conception Kit™ components out of reach of children.

### WARNING

Conception Cap is not intended for contraceptive use. Safety and effectiveness of the Conception Kit's™ Cervical Cap for contraception have not been established.

### CAUTION

 To avoid cervical irritation or infection, do not share or reuse the Conception Cap. Infection may endanger future fertility.

### CAUTION

Do not insert the Conception Cap for practice. Practicing insertion prior to actual use can cause a hostile environment within Conception Cap which could lead to infection and could impede pregnancy.

### CAUTION

In the following situations you should consult your health care professional:

If you are not sure about insertion and placement of the Conception Cap.

If you experience any discomfort or pain while the Conception Cap is in place.

If, at a time other than menstruation, upon removal there is blood on the Conception Cap. If you are unable to remove the Conception Cap.

**NOTE:** Washing Conception Cap with soap and water can prevent pregnancy.


**NOTE:** ② The Conception Cap is to be used no earlier than 24 hours and no later than 48 hours after detecting the LH surge. Normally ovulation will take place 24-48 hours after the LH surge.



## Using the Conception Cap (STEP 15):





1. Following sex, squeeze the semen from the semen collector into the Conception Cap. The semen will fill up to one third of the cap. 1 to 5ml is a normal amount of semen.

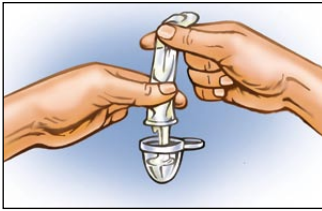


2. Pinch the cap together.



3. Insert the cap into your vagina and place it on your cervix.

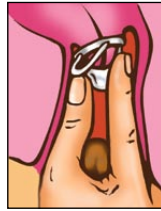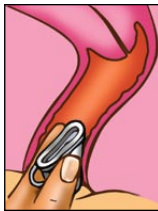

4. Leave the cap in place for at least 6 to 8 hours or overnight.
5. Go about your normal activities. Movement is not an issue.
6. Remove the Conception Cap. With your index finger, reach up, grab the loop and pull.

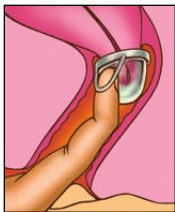

7. Discard the Conception Cap by placing it in the tray and throwing it away.

## STEP 16: USE THE PREGNANCY KIT.

### **⚠ WARNING**



To avoid choking hazard, keep all Conception Kit™ components out of reach of children.

### **⚠ WARNING**


**IVD** Pregnancy test kit is for in-vitro diagnostic use only. Not to be taken internally.

**NOTE:**  Do not reuse the test device.


**NOTE:**  (Do not use the pregnancy test kit after expiration date).

**NOTE:** 2C  30 C (35 F  86 F) Pregnancy test kit can be stored at temperatures of between 2 and 30 degrees Celsius or 35 to 86 Fahrenheit. Pregnancy test should be kept away from direct sunlight, moisture and heat.

**NOTE:** Do not stream (urinate) past the arrow on the home pregnancy test device.

**NOTE:**  The following is ONLY a summary of how to use home pregnancy test device. For more information, read and follow all instructions in manufacturer's pregnancy test insert found in the box.

### **Summary of how to use the home pregnancy test (STEP 16):**

1. On the first day after your missed period, use the pregnancy test to determine if you are pregnant.
2. Remove the home pregnancy test device from the sealed pouch and use it as soon as possible.
3. Hold the handle of the device in one hand. Use the other hand to remove the cap. You will want to save the cap for the next step so just put it aside for now.
4. Point the absorbent tip of the device downward, place in urine stream for at least 10 seconds to thoroughly wet.
5. After 10 seconds re-cap the device and wait for color bands to appear. Positive results may be observed in as little as 40 seconds. However, to confirm negative results, the complete reaction time of 5 minutes is required. Do not read results after more than 30 minutes.
6.  Reading the predictor: refer to the manufacturer's home pregnancy test insert inside the box on how to read the color bands to determine if you are pregnant.
7. If the test strip does not show results in the first 10 minutes, then you are not pregnant and you should continue to use the Conception Kit™. If the test strip indicates you are pregnant, it is time to contact your health care professional.


### **Care for the Conception Kit™**

**NOTE:** 2 C ↓ 30 C (35 F to 86 F) The Conception Kit™ can be stored at temperatures of between 2 and 30 degrees Celsius or 35 to 86 Fahrenheit. The Conception Kit™ should be kept away from direct sunlight, moisture and heat.

## **Conception Cap, Ovulation Predictor and Pregnancy Test are manufactured by:**

 Conceivex Inc.,  
Saranac, Michigan 48881 USA

**CE 0459**

 MediMark® Europe Sarl.  
11, rue Emile Zola. BP 2332  
38033 Grenoble Cedex 2.  
France

## **Semen Collector is manufactured by:**

 Durex Consumer Products, Inc.,  
Norcross, GA 30092 USA

## **Declaration of compatibility for the Conception Kit™**

Conceivex Inc. has verified the mutual compatibility of the devices in accordance with the manufacturers' instructions and has carried out our operations in accordance with the manufacturers' instructions.

Conceivex Inc. has packaged the Conception Kit™ and has supplied relevant information to users incorporating relevant instructions from the manufacturers in the individual components of the Conception Kit™ or in the Instruction Manual that accompanies each kit.

Conceivex Inc. ensures that all activities involved in the manufacturing and assembly of the Conception Kit™ are subjected to appropriate methods of internal control and inspection.

This declaration will be kept at the disposal of competent authorities.

Ovulation predictors:

 Conceivex, Inc.  
Saranac, MI 48881 USA  
Made in Canada

**CE 0459**

 MediMark® Europe Sarl.  
11, rue Emile Zola. BP 2332  
38033 Grenoble Cedex 2.  
France

Semen collector:

 Durex Consumer Products, Inc.,  
Norcross, GA 30092 USA

**CE 0459**

 MediMark® Europe Sarl.  
11, rue Emile Zola. BP 2332  
F-38033 Grenoble Cedex 2. France

Distributed by:  
Conceivex, Inc.  
Saranac, MI 48881 USA

The sperm-friendly intimate moisturizer sample:

 Ing Fertility  
17206 Spangle Creek RD  
Valleyford, WA 9906



www.mba-gbm.com   
+44(0)7710039721

Distributed by:  
Conceivex, Inc.  
Saranac, MI 48881 USA

The Conception Cap:

 Conceivex, Inc.  
Saranac, MI 48881 USA

 0459



MediMark® Europe Sarl.  
11, rue Emile Zola. BP 2332  
38033 Grenoble Cedex 2. France

Pregnancy test kit:

 Conceivex, Inc.  
Saranac, MI 48881 USA  
Made in Canada

 0459



MediMark® Europe Sarl.  
11, rue Emile Zola. BP 2332  
38033 Grenoble Cedex 2.  
France

